Ovulation-LH

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Ovulation-LH

Easy to use rapid Self-Test for the determination of the luteinizing hormone (LH) peak level in urine.

INSTRUCTIONS FOR USE

LH HORMONE AND OVULATION

Ovulation corresponds to the release of the ovum from the ovary. The increasing luteinizing hormone (LH) concentration induces the ovulation within 24 to 40 hours. This period is the most favourable to conceive a child. The PRIMA Home Ovulation Test is a rapid chromatographic immunoassay for the qualitative determination of LH in urine. In less than 10 minutes, it is possible to detect the LH peak preceding ovulation, and therefore to determine the days of optimum fertility.

CONTENT

The box contains:
- 5 sealed aluminium pouches containing 1 strip Ovulation LH Test and 1 desiccant bag*;
- 5 plastic cups for urine sampling;
- 1 instructions leaflet.
* the desiccant bag should not be used.

PRECAUTIONS

1. Carefully read the instructions before performing the test. The test is only interpretable if the instructions are carefully respected.
2. Keep out of the reach of children.
3. Do not use after the expiry date printed on the label and on the protective pouch or if the pouch is damaged.
4. Follow strictly the indicated time.
5. Store between +4°C and +30°C. Do not freeze.
6. Do not re-use the strip.
7. External use only. DO NOT SWALLOW.
8. This test is exclusively intended to in-vitro diagnostic self-testing use.
9. Consult a doctor in case of irregular or abnormally long cycles.
10. Do not use urine of pregnant, menopausal or hormone-treated women.
11. After use, all the components can be discarded in a dustbin.

WHEN PERFORM THE TEST?


The menstrual cycle duration corresponds to the number of days between the first day of menstruation and the first day of next cycle menstruation. The first day of menstruation is noted day n°1.

If the cycles are not regular, you must take into account the shortest cycle duration of the last 6 months. The table on the side will help you to determine the day when you must begin the test.


If your cycle is inferior to 21 days or superior to 40 days, please consult a physician. The test may be performed at any time of the day, but some women obtained better results with urine collected in the afternoon. If necessary, keep on performing the test everyday about the same time until detecting the LH peak. Do not use the first morning urine.

CYCLE DURATION	BEGIN OF THE TEST
21-22 days	6th day
23-24 days	8th day
25 days	9th day
26 days	10th day
27 days	11th day
28 days	12th day
29 days	13th day
30 days	13th day
31 days	14th day
32 days	15th day
33 days	16th day
34 days	17th day
35 days	18th day
36 days	19th day
37 days	20th day
38 days	21st day
39 days	22th day
40 days	23rd day

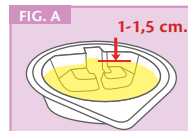PROCEDURE

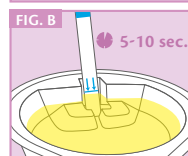1. Collect some urine in the supplied plastic cup (urine level: about 1 to 1,5 cm in the cup). - FIG. A
2. Tear the protective pouch (from the notch) and remove the strip only, holding it by the grip end. Dispose of the small desiccant pouch.
3. Dip the strip vertically in the cup containing your urine sample, placing it in the notch provided for that purpose during 5 to 10 seconds. - FIG. B

N.B. THE URINE LEVEL MUST NOT EXCEED THE HORIZONTAL BLUE LINE PRINTED ON THE STRIP UNDER THE SMALL ARROWS

4. Place the strip horizontally on a flat dry surface and allow the reaction to occur.

RESULT INTERPRETATION

READ THE RESULT AFTER 5 MINUTES.
DO NOT READ THE RESULT AFTER 10 MINUTES.

To interpret the result, it is necessary to compare the coloration of the Test band (T) with that of the Control band (C).

POSITIVE RESULT

The intensity of the Test band coloration is greater than the Control band's. This result means that the ovulation will occur within the next 40 hours and that if you have sexual intercourse within that period, you will optimise the opportunities for being pregnant.

NEGATIVE RESULT

The intensity of the Control band coloration is greater than the Test band's. This result means that the ovulation will not occur within the next 40 hours. In this case, you should perform another test the day after. Caution: in some cases, the Test band may not develop at all.

INVALID RESULT

If there is no Control band visible on the strip, it is not possible to interpret the test, which must be considered as inconclusive. In this case, it is recommended to repeat the test with a new Ovulation Test and a fresh urine sample.

F.A.Q. - QUESTIONS AND ANSWERS

How does the PRIMA Home Ovulation Test - LH work? In women, the average hormonal cycle lasts 28 days. The FSH (Follicle Stimulating Hormone) secretion induces the follicle maturation, and LH (Luteinizing Hormone) secretion induces ovulation. During the menstrual cycle a small quantity of luteinizing hormone (LH) is secreted, but in the middle of the cycle the LH concentration increases very rapidly. The LH peak only appears 12 to 18 hours before ovulation. Ovulation corresponds to the release of the ovum by the ovary. The ovum may be fertilised within the 12 to 24 hours that follow its release: this period is the most favourable to the conception. Therefore, conception is more likely to occur within the 24 to 40 hours following the LH peak. The PRIMA Home Ovulation Test is a test for the determination of LH peak in urine.

I do not know the usual duration of my cycle, or the duration varies from one cycle to the other. How could I know what day I should begin the test? If you do not know your usual cycle duration, we advise you to wait for a total cycle before starting the test. If your cycle varies with more than 3 days, take into account your shortest cycle of the last 6 months and use its duration to determine the day of the rest.

What is the best moment to use the test? You can perform the test at any moment of the day; however, the urine that is collected at the end of the afternoon contains more LH which will enable to obtain better results. In any case, perform the following tests every day about the same time. Do not use the first morning urine.

Can the result be incorrect? The results are accurate insofar as the directions for use are scrupulously followed. Nevertheless, the result can be incorrect if the test is damp before performing the test. Moreover, if the test is used by a pregnant or menopausal woman, or by a hCG or LH-based hormonal treatment, or by a woman taking pill, the results can be incorrect. In case of unexpected or contradictory results, it is recommended to consult a gynaecologist, for some health problems can induce incorrect diagnosis.

I left the absorbent tip in urine for more than 10 seconds. Will I get an accurate result? Yes. A few more seconds will not affect the results.

I left the absorbent tip in urine for less than 5 seconds. Will I get an accurate result? No, because the sample volume might be insufficient.

What should I do if the test is positive? If the result is positive, you have detected the LH peak, which means that the ovulation will occur within the next 40 hours. If you have sexual intercourse within this period, you will optimise the opportunities for getting pregnant.

What should I do if the test is negative? If the result is negative, you have not detected the LH peak, which means that the ovulation will not occur in the next 40 hours. A new test should be performed on the following day.

Should the test be used as contraception method? No, the test has not been designed for this purpose and should not be used for this purpose.

Will discontinuing my hormonal contraception (pill) affect the result? If you use a contraceptive and that you have just discontinued it, it is possible that your cycles may be irregular for some time, which may affect the result. It is recommended to wait for two cycles before using the Ovulation tests.

I have been using the PRIMA Home Ovulation Test during several months, but I have not got pregnant yet. May I be sure to get pregnant one day? The Ovulation test is designed to determine your more fertile days. If you had sexual intercourse during that period, you may get pregnant, but other reasons may explain why you are not pregnant yet. In this case, it is recommended to consult a physician or gynaecologist.

I have just performed a test and the result is positive for the first time, what is the best period to get pregnant? In order to optimise the opportunities to get pregnant, it is recommended to have sexual intercourse on the day of the peak apparition and/or the following day.

What day should I stop performing the test? You can stop performing the test as soon as the result is positive. The unused tests can be kept for the next cycle.

Should I use all the tests? No, you can stop performing tests as soon as you have identified your LH peak day, and keep the unused tests for another cycle if necessary.

What is the accuracy of PRIMA Home Ovulation Test? PRIMA Home Ovulation Test is very accurate and has been used for more than 10 years by professionals in the field. Evaluation reports show an overall agreement of 100% with reference method.

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